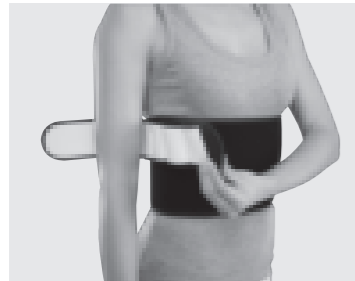
1

2

3

4

Deutsch

medi Arm fix

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medi Arm fix

Intended purpose

The medi Arm fix is a shoulder immobilisation support. The product is exclusively to be used for the orthotic fitting of the shoulder and only on intact skin or if any wounds have been properly covered.

Indications

- Conservative and postoperative treatment following proximal fractures of the head of the humerus
- Conservative and postoperative treatment of fractures of the scapula
- Conservative and postoperative treatment of AC joint dislocation
- Conservative therapy of painful osteoarthritis of the shoulder
- Conservative and postoperative treatment of impingement syndrome
- Conservative and postoperative treatment of osteoarthritis of the AC joint
- Postoperative treatment after shoulder replacement
- Conservative treatment after dislocation of the shoulder
- Conservative treatment of chronic dislocation of the shoulder
- Conservative and postoperative treatment of fractures of the distal humerus

Contraindications

Conservative and post-operative therapy of posterior shoulder dislocation

Fitting instructions

1. Wrap the wider strap around your body and fasten it just below the chest.
2. Fix the injured arm with the strap around the upper arm. Attach the lower arm sling to the abdominal strap. Open the sling and place the

injured arm inside. Close the sling afterwards.

3. Pass the thin strap over the healthy shoulder and attach it to the lower arm sling. Adjust the length of the strap to the required length.

Care instructions

Hook and loop fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Polyamide, polyester

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

Your medi team

wishes you a quick return to full fitness.

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